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If you plan to travel by air with your insulin pump, some advance planning may make your security screening go smoother

Your device is safe for use on commercial airlines and complies with FAA wireless transmission standards.* It is also designed to withstand common electromagnetic interference and can be safely carried through metal detectors.

However, your Tandem Diabetes Care insulin pump should NOT be put through machines that use X-rays, including airline luggage X-ray machines and full-body scanners.

We recommend disconnecting at the infusion site and asking the security agent for an alternative screening method. If you prefer to stay connected, you can notify the agent about your pump and request to go through a standard metal detector wearing your pump.

It may be helpful to provide a printout of the information below to the security agent during the screening.

If you have any questions about traveling with your insulin pump, please visit die-clevere-insulinpumpe.de or call the VitalAire Customer Support. 0800-100 16 44 (free of charge from a German landline) or +49 (0)421-48 63 18 63 (from abroad) We are here for you 24 hours a day, 7 days a week.

* The pump complies with radiated emissions requirements defined in RTCA/DO-160G, Section 21, Category M.

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Dear TSA or airport representative,


This is an insulin pump (t:slim X2™ insulin pump) from Tandem Diabetes Care, a medically necessary medical device prescribed by a physician. It is designed to withstand common electromagnetic interference and can be safely carried through metal detectors, but it should not be exposed to any form of X-rays. This includes airline luggage X-ray machines and full-body scanners.

Please provide a screening method other than X-ray for this insulin pump.

If you have any questions about this device, please call VitalAire Support at +49 (0)800-100 16 44 (24/7 free of charge from German landline) or +49 (0)421-48 63 18 63.

Sincerely,
VitalAire

VitalAire GmbH · Bornbarch 2 · 22848 Norderstedt · Germany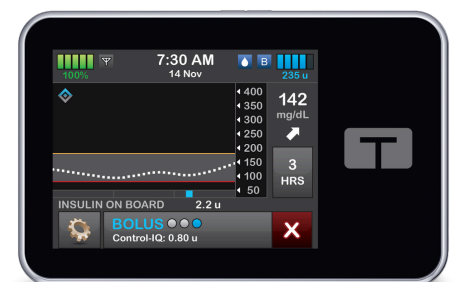

- ✓ Metal Detectors
- ✗ X-Rays,
including full body scanners and luggage screening machines